

CLEANING & STERILISATION OF EQUIPMENT

Avoid needle stick injury – wear utility gloves & handle dirty equipment with care

STEP 01 ► RINSE

With warm running water

◀ To loosen dirt & debris

STEP 02 ► WASH

In warm water & detergent using an instrument brush or place in an ultrasonic cleaner, activate for 10–15mins

◀ To remove dirt & debris

Use low alkaline low foaming instrument grade detergent

Cotton buds, toothbrushes, pipe-cleaners & hand towels should **NOT** be used to clean equipment as they can shed lint or bristles, damage equipment & prevent sterilisation

STEP 03 ► RINSE, DRAIN & DRY

With hot running water - run water through tubes & hollow equipment
Dry using a low lint cloth

◀ To remove remaining detergent, dirt & debris

STEP 04 ► STERILISE

Pack clean dry equipment for sterilisation

◀ To kill all germs & spores