

# D95 P2 Respirator Mask

The **D95 P2 Respirator Mask** is designed to achieve a comfortable and adequate seal by joining the mask's two bands with the FitClip™. The Regular FitClip™ is attached as standard, and where a longer band is required, the XL FitClip™ can be ordered separately.



## Donning the Mask

Prior to donning, ensure hand hygiene is undertaken.



1. Move the FitClip™ to the band on your preferred side for donning.
2. Open the respirator mask, creating a gentle curve in the nose wire.
3. Lean chin into the mask and lift bands toward the crown of the head.
4. Secure straps on the FitClip™. Adjust mask bands for a secure and comfortable fit.

## DoFFing the Mask Safely

Prior to doffing, ensure hand hygiene is undertaken.



1. Without touching the mask, follow bands towards the FitClip™ and gently unhook, keeping tension on the bands.
2. Lean forward, maintaining control of the band tension, whilst slowly removing the mask away from the face. Ensure the mask does not flick forward or back onto the face.
3. Discard the mask safely. Ensure hand hygiene is performed once the mask has been discarded.

## Fit Check

A fit check **MUST** be performed **EVERY** time a D95 P2 Respirator is worn.



5. Firmly place one finger on the centre of the nose wire.



6. Apply thumb and first finger to either side of the nose. Pinch in along the nose piece and onto the face.

**Note:** A firm pinch around the nose is required to achieve a seal. Avoid creating a "peak" in the nose piece.



7. Once the mask is firmly moulded to the nose, squeeze the nose piece and gently drop the jaw, allowing the mask to fit the natural shape of the face.



8. Look left and right, then up and down. There is to be no movement of the mask at the nose wire. Inhale and exhale deeply checking for escape of air.



9. If there is any movement of the mask at the nose wire, or escape of air, repeat steps 5 onwards.



If you have not achieved a successful fit check by following steps 5-9, you may have an incorrectly fitted D95 P2 Respirator which cannot provide the intended level of protection.

You **MUST** seek advice from your local worker/staff health or infection control unit in your facility.



Scan here for video demonstration.

[detmoldmedical.com/detmed/resources/videos/](https://detmoldmedical.com/detmed/resources/videos/)

**Caution:** Masks help protect against certain particles and are designed for occupational use by healthcare professionals who are properly trained in mask use and limitations. Misuse may result in sickness or death. DO NOT use in any manner not indicated in the Instructions for Use. Always read the label. Discard the mask if it becomes damp, soiled, damaged, or difficult to breathe through. Performance efficiencies may deteriorate when exposed to oil-based substances or individual fit characteristics are not adequate. Do not use if the packaging is damaged. Maximum operating temperature is +50°C.

✉ [connect@detmoldmedical.com](mailto:connect@detmoldmedical.com) 📞 1800-DET-MED (1800 338 633)